



# Igenity® Hair Sample Instructions



1. Pull (do not cut) hair from the tail switch, not tail head. Pull hair in the opposite direction from which the hair is laying. This results in less breakage than pulling straight out from the tail.



3. Roots must be clearly visible. Approximately 30 hair roots are needed. For animals with finer hair, a minimum of 50 to 60 hairs is desired. For animals younger than three months, blood is recommended since hair roots will likely not be visible.



5. Trim excess hair extending from the collector. Write animal ID and other information in the spaces provided.



2. Make sure hair is free of fecal material and dirt.



4. Open the collector and peel back the plastic cover. Insert the root end of hairs in the middle of the collector. Seal the plastic cover over the hair and then close the collector cover tab.



6. Properly record animal IDs on the collector cards and correctly enter that information on your testing order form to ensure you are matching the right DNA to the right animal.



Neogen: GeneSeek Operations • 4131 N. 48th Street, Lincoln, NE 68504  
877-443-6489 • [www.neogen.com/genomics](http://www.neogen.com/genomics) • [igenity.support@neogen.com](mailto:igenity.support@neogen.com)

